

Lava™ All-Ceramic System


CLINICAL PREPARATION

To assure clinical success, the following minimum material thickness is recommended.

Preparation:


Anterior Crown

- 1.5 2.0 mm incisal reduction
- 1.0 1.5 mm labial and lingual reduction
- Round the internal line angles
- · Chamfer margin

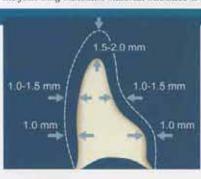


Posterior Crown


- 1.5 2.0 mm occlusal/incisal reduction
- 1.0 2.0 mm axial reduction
- Round the internal line angles
- · Chamfer margin

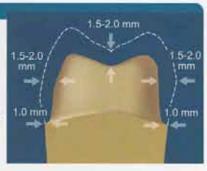

Preparation:


The optimal preparation is a shoulder or chamfered preparation with a circumferential step or chamfer which must be applied at an angle of >5° (horizontal). The angle of the preparation (vertical) should be 4° or larger. The inside angle of the shoulder preparation must be given a rounded contour.

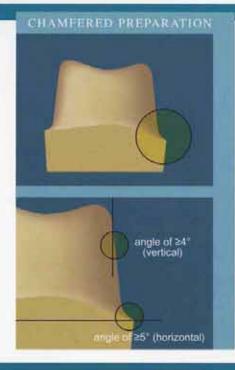


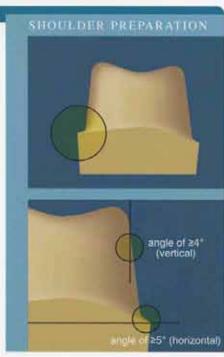

Contraindication:

We do not recommend a feather edge, as it does not provide adequate reduction for the porcelain buildup, or the trough (or gutter) shoulder because the outer most edge may not be detected when scanned.

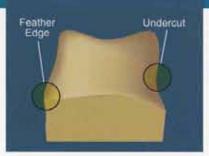

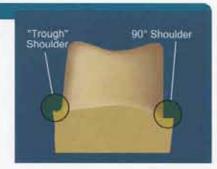



Dental Products

3M Center Building 275-2SE-03 St. Paul, MN 55144-1000



Recycled Paper 40% Pre-consumer waste paper 10% Post-consumer waste paper Printed in U.S.A. Copyright © 3M 2002. All rights reserved. 70-2009-3508-1

